



XO[®] 4-1

Quick Guide

Switch on/off XO 4 with the green main switch on the rear of the unit stand.

When a unit instrument is lifted forward XO Foot Control manages this instrument - see next page.

When all unit instruments rest XO 4 Unit and Chair functions are managed with XO Foot Control and XO Joystick.


Operate the *cuspidor* manually either with  or with XO Foot Control .

Activate **cup filler** with  to add water.



Suction starts automatically when hoses are lifted from holder.


Manage **XO 4 Lamp** with no touch sensor under lamp head.

Hold your hand within activation distance for less than 1 second and lamp switches on/off. Hold hand for more than 1 second to adjust light intensity.


Use  to **call dental assistant**.

Unit and Chair

Bring **XO 4 Chair** to Position 1 with  and to Position 2 with .

First time  is activated chair goes to Rinse Position.

Second time  is activated chair goes to Zero Position.

If  is activated twice within 1 second chair goes directly to Zero Position.

Go to Last Position with .


Emergency stop chair: Stop chair immediately while moving by touching any switch on XO Foot Control or XO Joystick (except .




 and  switches.





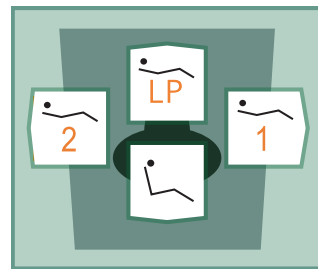



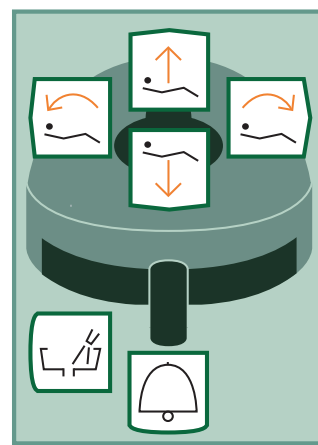


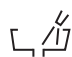

No touch sensor.



XO Joystick on chair base.





XO Foot Control.

-  Working Position 1
-  Working Position 2
-  Once: Rinse Position
Twice: Zero Position
-  Last Position
-  Lift chair
-  Lower chair
-  Back rest: Tilt forward
-  Back rest: Tilt backward
-  Call dental assistant
-  Start & stop cuspidor flush
-  Start & stop cup filler

Micro Motor



Lift *micro motor* forward and control it with XO Foot Control.


Run motor clockwise with  or counter clockwise with  in the range 250 – 40.000 rpm.

Change spray selection: water & air, air only or no spray with .

Turbine


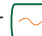
Lift turbine forward and control it with XO Foot Control.

Start/stop turbine with  or .

Change spray selection: water & air or no spray with .



Ultrasonic Scaler

Lift scaler forward and control it with XO Foot Control.

Start/stop oscillation (power) of scaler with  or .




Switch cooling irrigation on/off with .

Choose between 3 power settings:

Increase level with 
Decrease level with .

See current power setting on display:

-  Minimum level
-  Medium level
-  Maximum level

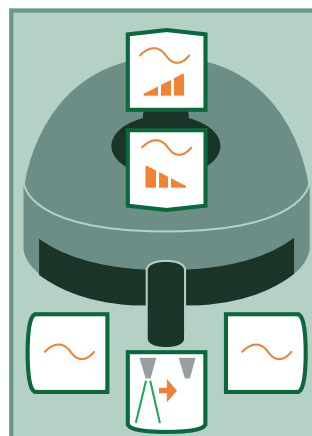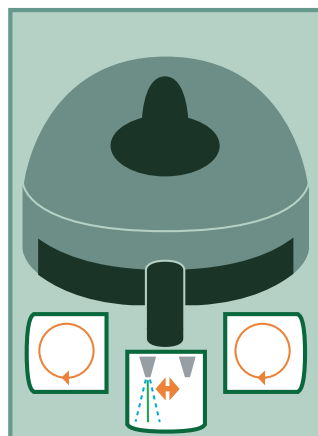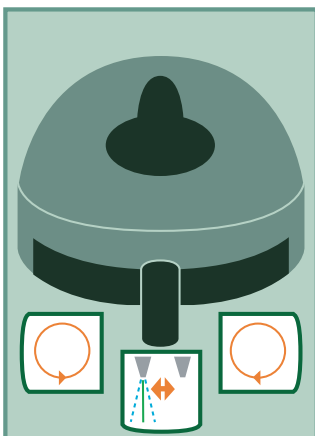

Light Curing Lamp

Lift light curing lamp forward and control it with XO Foot Control.

Exposure time is preset to 20 seconds.

Start curing process with  or .


Hear a beep after each 10 seconds.

Intraoral Video Camera





Unlike all other unit instruments you may use video camera also when another instrument is active.



Lift camera forward and display a video image on a screen.







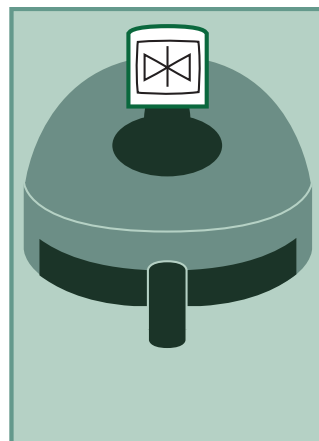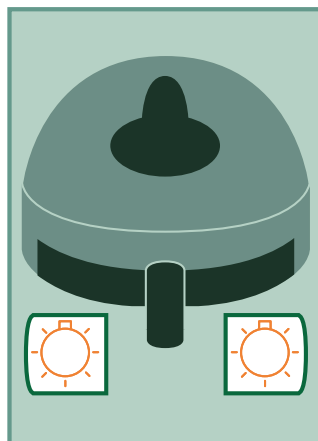


Control video camera with XO Foot Control when no other instruments are active.

Toggle between "still" and "live" mode with .

Note: "Still" function requires installation of appropriate software on connected PC.

-  Spray air & water
-  Spray water
-  Spray air
-  No air or spray water.



-  Start rotation counter clockwise
-  Start rotation clockwise
-  Spray selection
-  Power on Scaler
-  Irrigation on/off Scaler
-  Increase maximum power / Scaler
-  Decrease maximum power / Scaler
-  Start LC Lamp
-  Still / Live mode Video Camera

Maintenance

Infection Control Routines

If XO Foot Control slides on floor when activated:

Clean rubber feet under foot control with benzine.

Important note:

After disinfecting the instrument bridges' surface, please do not replace the rubber protection pad until the surface is completely dry.



Important note:

In the event that the instrument hoses have been removed, one must ensure that the plugs and sockets are completely dry before the hoses are re-mounted on the instrument bridge again.




XO CARE A/S
 Usserød Mølle
 DK 2970 Hørsholm
 Denmark
 +45 70 20 55 11
 info@xo-care.com
 www.xo-care.com



In the morning before first patient

1. Finish water line disinfection procedure:
 - Check that unit instruments are still in instrument holder on cuspidor
 - Switch on unit
 - Unit water lines are automatically flushed with disinfection liquid
 - See status on display
2. Fit sterile / disinfected / new:
 - Unit instrument pad
 - Unit instruments
 - Cup
 - Handles on instrument bridge, lamp and screen
 - Suction filters and covers for suction nozzles.

After each patient

1. Remove contaminated:
 - Unit instruments
 - Unit instrument pad
 - Cup
 - Handles on instrument bridge, lamp and unit screen
2. Clean and disinfect surfaces
3. Fit sterile / disinfected:
 - Unit instrument pad
 - Unit instruments
 - Cup
 - Handles on instrument bridge, lamp and screen
4. Fit disinfected:
 - Cuspidor bowl, gold trap and cup holder. (After placing the cuspidor bowl, turn the cuspidor faucet back to the right position)
5. Start water line disinfection procedure:
 - Place all unit instruments in instrument holder on cuspidor
 - Press  to empty expansion container
 - Switch off unit leaving unit instruments in instrument holder on cuspidor

In the evening after last patient

1. Disinfect suction system:
 - Activate 
 - Fit both suction hoses on disinfection nipples
 - Activate 
 - See status on display
2. Remove contaminated:
 - Unit instruments
 - Unit instrument pad
 - Cuspidor bowl, gold trap and cup holder (turn the cuspidor faucet before moving the cuspidor bowl)
 - Handles on instrument bridge, lamp and screen
 - Suction filters and covers for suction nozzles
3. Clean and disinfect surfaces and chair upholstery



Water line disinfection.



Suction hoses fitted at disinfection nipples.



Removal of cuspidor bowl.



XO 4 parts, which are autoclavable (134 °C).



Parts, which may be washed in a thermo-disinfector (90 °C).



Clean rubber feet under XO Foot Control regularly with benzine.