

---

XO<sup>®</sup> 4

Retrofitting Dörr CS 1 Combi-Sepamatic. XO 4.

---



YB-788  
Version 1.00

XO CARE A/S  
Usserød Mølle  
Håndværkersvinget 6  
DK 2970 Hørsholm  
Denmark

+45 70 20 55 11  
info@xo-care.com  
www.xo-care.com

**XO**<sup>®</sup>  
Free to focus<sup>™</sup>


This page is left blank.

The complete kit XO-210




J12 connector.  
24V power supply.


Mount the 24V power-supply on the wall right next to the 230V AUX output J12 where connection must be.

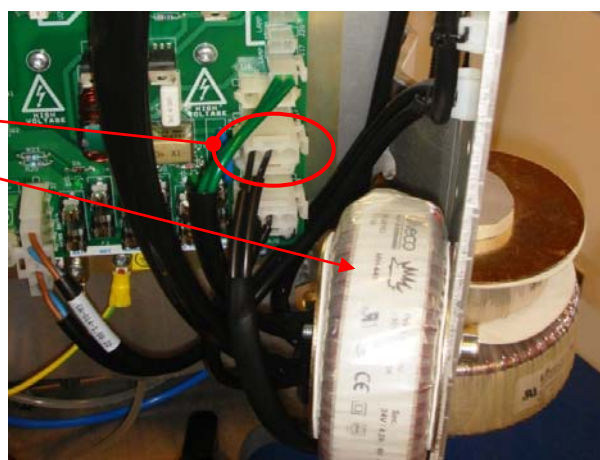

In the top of the backplane find the connector J8 connect the CA-035 signal cable.

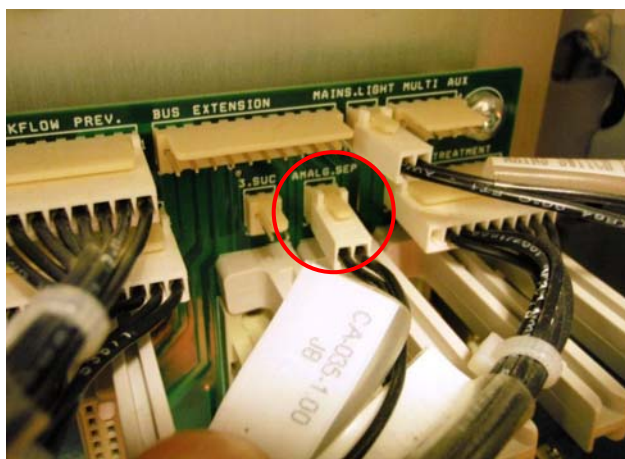


Draw the two cables to the mount-area,  
located under the select-vac,  
(Draw together with the other cables under  
top plate)

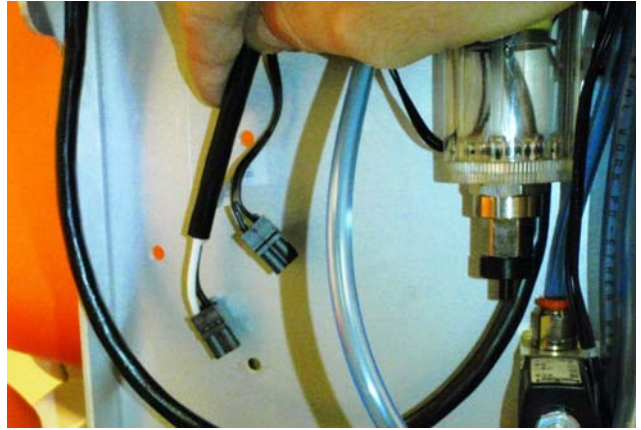


Mount the CS1 motor on the wall.

Tool size Torx 20.  
Suction valve.  
Signal cable CA-035.  
Power 24V from trafo.

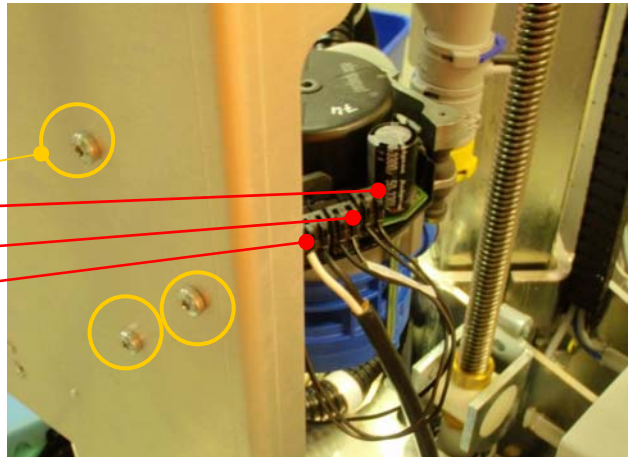


Water lock system to fountain and cup filler drain.

Ventilation hose goes to the top  
Note: Must be secured with cable-tie.

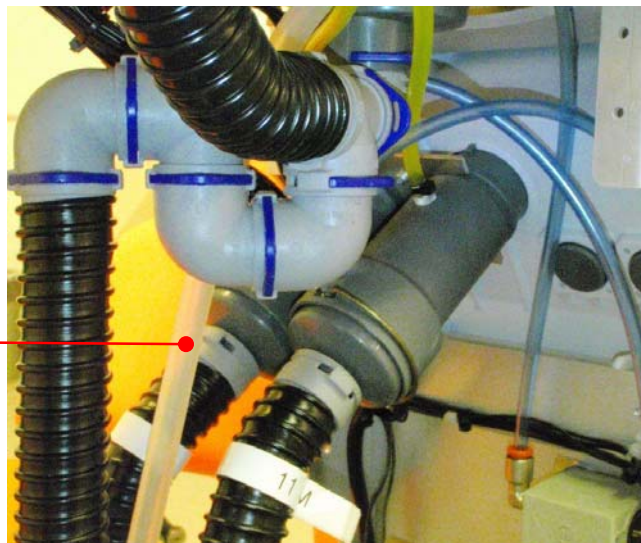

- Hose to return air.
- Vacuum hose for select vac.
- Suction.
- Drain.

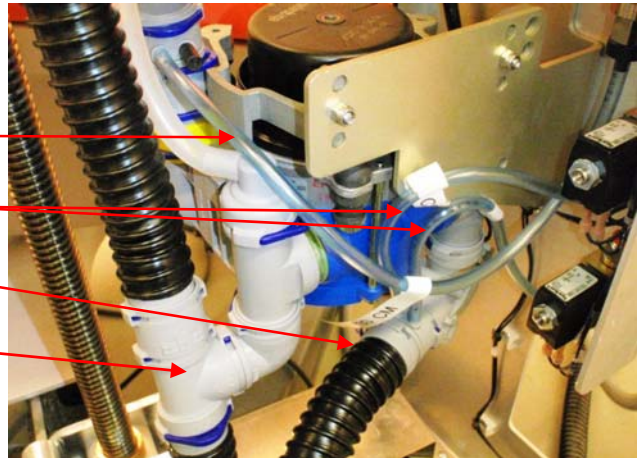

Check CS1 functionality before re-assembling the unit again.

